

**STORAGE CONDITIONS:**

Store at ambient temperature. Keep in a dry and cool place, avoid excessive heat.

**SHELF LIFE:**








The SureKlot™ Spray has a shelf life of 2 years.

**HOW SUPPLIED:**

- 100 ml - SKVS100



If you have any questions or comments about the product, please contact Axio Biosolutions Helpline at [info@axiobio.com](mailto:info@axiobio.com) or +91 8860786067

	Keep away from sunlight
	Keep dry
	Do not use if pack is damaged
	Manufacturer
	Date of manufacture
	Expiry date
	LOT number

SKS/FU/13.07.23/Rev01

**Sure**<sup>TM</sup>  
**Klot**  
SPRAY

**INSTRUCTIONS FOR USE**

 **AXIO**<sup>®</sup>

**Manufacturer:**

Axio Biosolutions Private Limited  
Plot 18, Gujarat Pharma Techno Park  
Sari Matoda, Sanand Taluka  
Ahmedabad-382220, Gujarat, India.  
Mfg License no: MFG/MD/2020/000102  
Customer Care: +91 88607 86067  
Email: [info@axiobio.com](mailto:info@axiobio.com) | Web: [www.axiobio.com](http://www.axiobio.com)

## SureKlot™ Spray

### DEVICE DESCRIPTION:

The **SureKlot™ Spray** is a clear, non-viscous, and nearly odourless liquid intended to be used for cleansing, decontamination, and moisturization of external veterinary wounds. The cleansing action of the solution helps in the removal of foreign materials such as dirt and debris from the wound surface.

### COMPOSITION:

**SureKlot™ Spray** is composed of chitosan lactate in an aqueous base with preservative (0.1 % Polyhexamethylene biguanide).

### ACTION/BENEFITS:

The **SureKlot™ Spray** formula of chitosan and PHMB cleans the wound and gently removes biofilm. It has strong cleansing action, yet is gentle on the wound surface, and creates the optimum environment for wound management. It effectively removes foreign materials, dirt and facilitates the wound debridement process. It is non-cytotoxic, non-irritant and non-staining.

### INTENDED USE:

The **SureKlot™ Spray** is intended for the cleaning, decontamination, and moisturization of external veterinary wounds. It also assists in autolytic debridement and removal of bacterial biofilm.

### INDICATION FOR USE:

The **SureKlot™ Spray** is indicated for the management of chronic and acute external veterinary wounds. It can be used for cleansing wounds and for moistening and lubricating absorbent wound dressing of the following wounds

- Infected veterinary wounds or wounds having an increased risk of infections
- Maggot-infested wounds
- Dermatomycosis

- Necrotic wound
- Pressure sores
- Leg ulcers
- Cavity wounds
- Traumatic and Surgical wounds
- Footrot
- Burns wound
- Minor cuts, bruises, abrasions

### DIRECTIONS FOR USE:

- Rinse the wound with an ample amount of the **SureKlot™ Spray** for cleaning and rinsing.
- For heavily contaminated wounds: Cover the wound with the **SureKlot™ Spray** for 10-15 minutes, using standard gauze saturated with **SureKlot™ Spray**.
- For best results, use along with the **SureKlot™ Powder & SureKlot™ Gauze**.

### ANTIMICROBIAL EFFECTIVENESS TESTING:

Antimicrobial Effectiveness Testing was demonstrated using USP <51>, with an expanded list of microorganisms, *Staphylococcus aureus*, *Escherichia coli*, *Pseudomonas aeruginosa*, *Candida albicans*, and *Aspergillus niger*.

**SureKlot™ Spray** successfully meets USP <51> category 2 criteria. No microbial growth was observed at 7, 14, and 28 days.

### WARNINGS:

- For external veterinary use only.
- Use within 28 days after opening.
- Do not apply to the eyes, mucous membranes.
- If redness, irritation, or swelling persists or increases, discontinue use and consult a veterinarian.